

Tips for keeping your skin healthy



Do

- Always clean your skin with warm water
- Change your pouch routinely
- Resize your template regularly
- Keep your routine simple
- Inspect your skin every time you change your bag
- Remove your pouch carefully
- Tell your Stoma Care Nurse if you are taking other medication
- Keep in touch with your Stoma Care Nurse



Don't

- Use soap or other solutions to wash your skin
- Wait for your pouch to leak
- Add in accessories to your stoma care routine
- Pull your bag off too quickly
- Ignore any changes in your skin around your stoma, however small
- Try and manage your stoma care problems alone – contact your Stoma Care Nurse

We would like to acknowledge the help and support of Jo Hoeflok (Enterostomal Therapy Nurse, Canada) in developing the content for this document.

The Hollister logo and "Healthy skin. Positive Outcomes." are trademarks of Hollister Incorporated.
© 2020 Hollister Incorporated.



Ostomy Care
Healthy skin. Positive outcomes.



Take A Look

Keeping Your Stoma

and Skin Healthy

Taking care of your stoma and the surrounding skin is important

We know that most people don't know when there is a problem. This **TAKE A LOOK** card will help you to check your stoma and your skin: it tells you what is normal and what is not normal.



Ostomy Care
Healthy skin. Positive outcomes.

Take a look at what's **NORMAL**



Your skin **should be:**

- Dry
- No rash
- No redness
- No flakiness


Take a look at what's **NOT NORMAL**



Does your skin have **one or more of these?**

- Rash
- Redness
- Wet or flaky
- Itchy
- Burning
- Painful

Your **TAKE A LOOK** tip

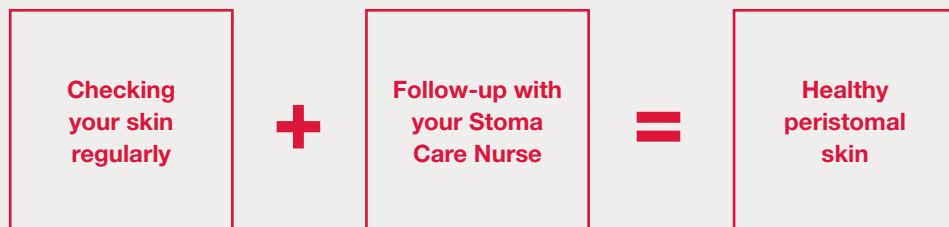

If you have any worries about your stoma or the skin around your stoma, call your Stoma Care Nurse to **TAKE A LOOK**.

Key Contacts and Customer Service

0800 521377

Please contact Customer Service during the hours of:
Monday - Friday: 8.30am to 5pm

Your Nurse:

.....

Telephone number:

.....